

LoFric[®]
Cath-Kit[™]

A convenient hydrophilic catheter system for CIC





Cath-Kit makes everyday life a little easier.

Portable convenience

Having freedom of movement in everyday life is often very important for your quality of life. If you are dependant on self-catheterization, finding a toilet may not always be easy. LoFric Cath-Kit is the LoFric catheter combined with a urine collection bag. Everything you need to self-catheterize without access to a toilet - just add water and you're ready to catheterize.

Proven clinical benefits


The catheter's surface layer binds water and is virtually frictionless upon insertion and removal. Compared with other catheters, the friction against the urethra is reduced up to 95%, offering reduced risk of UTIs and complications, and greater comfort. Because the kit is completely closed, the catheter maintains its unique chemical properties and the surface layer stays intact until it is ready to be used.




LOFRIC® CATH-KIT PRODUCT RANGE

TYPE	LENGTH	SIZE
LoFric Cath-Kit, Male	16", straight tip	8FR - 18FR
LoFric Cath-Kit, Male	16", coudé tip	10FR - 18FR
LoFric Cath-Kit, Female	8", straight tip	8FR - 18FR
LoFric Cath-Kit, Pediatric	8", straight tip	6FR - 10FR

CONVENIENT AND PORTABLE

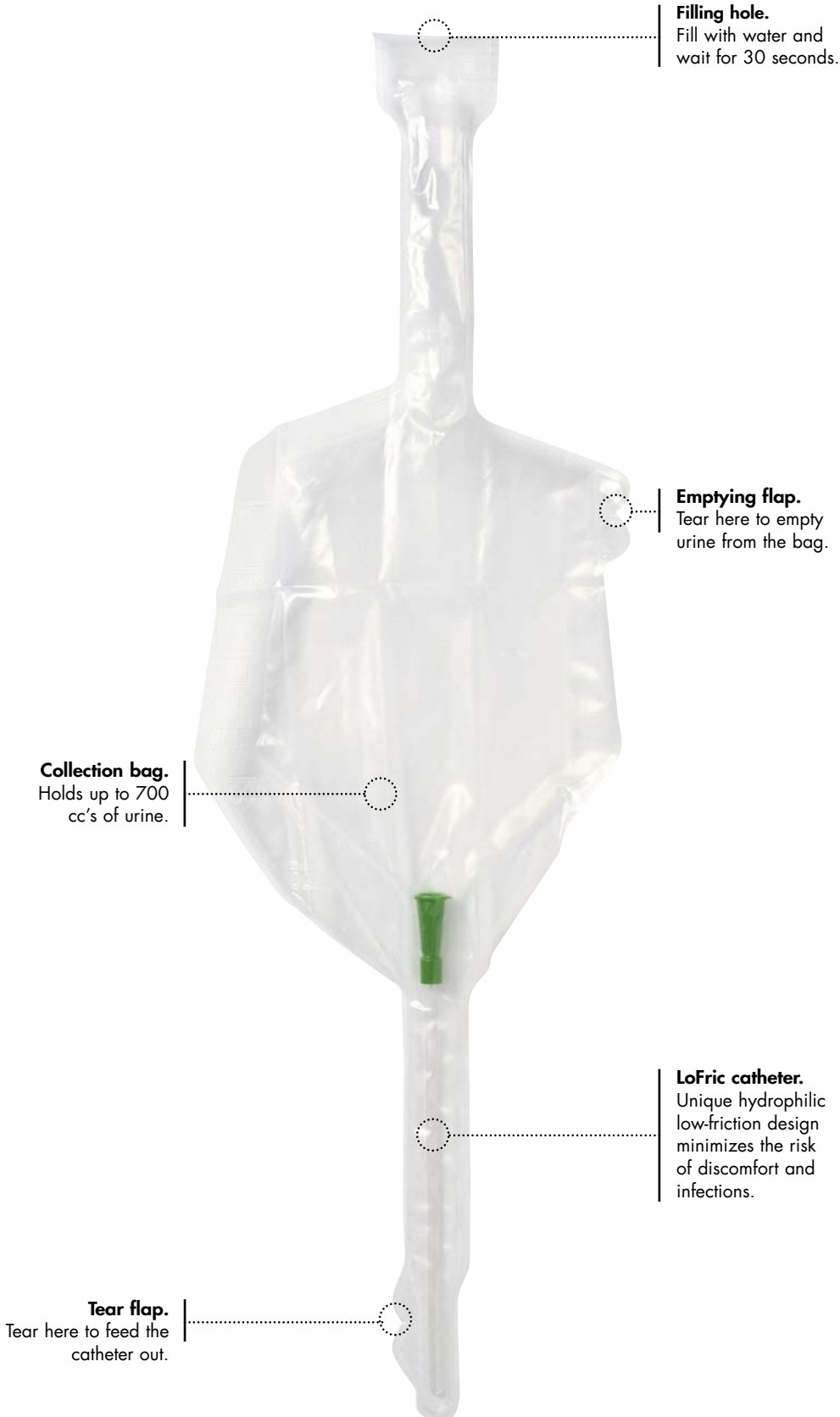

Directions for Use:

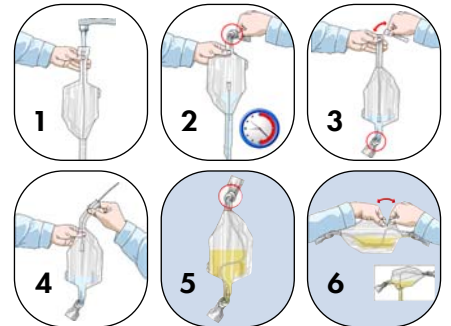

1. Hold the product upright as shown and add water to the 700 mark.

2. Push the catheter down into the tail of the bag and tie off the neck. Soak the catheter for at least 30 seconds.

3. Invert the bag and tear off at the flap where shown

4. Ease the catheter up through the tail of the bag as shown and catheterize.

5. After catheterizing, push the catheter back into the bag and tie off the tail.

6. The bag has a tear flap for emptying the contents prior to disposal, if necessary.

LoFric is designed for one-time use only.
After a single use, the hydrophilic coating is not assured to hold its lubricating water layer on inside the urethra. This may cause injury to the urethra and /or increase the risk of infection.

All components are 100% latex-free.

Knowledge, education and support

If you need additional information, we have comprehensive knowledge in all the areas associated with catheterization, dilatation, and instillation. We are willing to act as a resource, or to provide education.

When it comes to education and information, we have a broad program. You may use our training and information on video, in print, or on the Astra Tech website. Apart from that, we have information on products, clinical studies, user guides, starter kits, and much more. Contact us for all your needs.

