

The LoFric catheter is equipped with **Urotonic<sup>™</sup> Surface Technology** to give the lowest friction during insertion and removal.

No drainage eyes.



**100% latex free**

## For Safe and Easy Dilatation

---

### Intended Use

Hydrophilic single-use dilatation catheter for home or healthcare use  
Dilatation of the urethra

---

### Material Composition

Hydrophilic surface coating: poly(vinyl pyrrolidone)/PVP and table salt/NaCl  
Catheter: medical grade poly(vinyl chloride)/PVC  
Item sterile package: Lacquered paper, laminated film of polypropylene/PP and polyethylene/PE

---

### Package

25 catheters per box / 4 boxes per shipping case

---

### Sterilization & Shelf Life

All catheters are sterilized by ethylene oxide gas. Information about the recommended best-before date is printed on the package.

## Directions for Use:



1. Peel open the top of the package about 2 inches.
2. Fill the package with water until the water reaches about an inch below the funnel end.
3. Remove the blue patch at the top of the package and attach the pack to a dry surface.
4. Allow the catheter to soak for at least 30 seconds, then use.

**LoFric is designed for one-time use only.** After a single use, the hydrophilic coating is not assured to retain its water layer inside the urethra. This may cause injury to the urethra and/or increase the risk of infection.

### LoFRIC® PRODUCT RANGE

ITEM #	TYPE	SIZE
901640	LoFric, Male	16" 16FR
901840	LoFric, Male	16" 18FR

#### Catheter sizes

16FR	
18FR	